

Clinical Presentation <sup>1</sup>	What to use	When to use*	How much to use
No redness and skin intact (at risk)		 Day and night	 Pea-size increments
<b>Category 1</b> Red but skin intact		 Day and night	 Pea-size increments
<b>Category 2</b> Red with moderate to severe skin breakdown		 Day and night	 Apply an even coat

**\* Note**  
3M™ Cavilon™ Durable Barrier Cream and 3M™ Cavilon™ No Sting Barrier Film need to be applied after every 3-4 incontinent episodes. More or less frequent application may be required depending on the frequency of incontinent episodes.

**3M Australia Pty Limited**  
ABN 90 000 100 096  
Building A, 1 Rivett Rd  
North Ryde NSW 2113  
Phone 1300 363 878  
www.Cavilon.com.au

**3M New Zealand Limited**  
94 Apollo Drive  
Rosedale, Auckland, 0632  
New Zealand  
Phone 0800 80 81 82  
www.Cavilon.co.nz

AV011451950  
3M and Cavilon are trademarks of 3M Company. Please recycle.  
© 3M 2016. All rights reserved.

**Reference**  
1. Beekman D et al. Proceedings of the Global IAD expert panel. Incontinence-associated dermatitis: moving prevention forward. *Wounds International* 2015. Available to download from [www.woundsinternational.com](http://www.woundsinternational.com)



**Medical and Surgical Requisites Pty Ltd**

Call within Brisbane 07 3859 2900 33 Fulcrum Street Richlands Queensland 4077 www.medsurg.com.au  
Phone 1300medsurg (1300 633 787) Email medical@medsurg.com.au Fax 07 3859 2995

