

# 3M™ Surgical Clipper 9661

For use with 9660 blade



Unique pivot head  
**conforms**  
to body contours

- The only surgical clipper with a pivot head design
- Pivoting blade conforms to body contours, skin folds and creases
- Away-from-the-skin cutting blade minimizes potential for nicks or cuts
- Has a nick rate of <1%, compared with other clippers' higher nick rates\*
- Accommodates flat and angled clipping styles
- Cuts through all types of head and body hair in a single pass
- Clean by rinsing or wiping with disinfectant
- Also available, the 3M™ Surgical Clipper Specialty Blade 9690 for head and heavy body hair removal

## Ordering Information

Catalog Number	Product Description	Items/Box	Boxes/Case
9660	3M™ Surgical Clipper Blade 9660	50	1
9661	3M™ Surgical Clipper 9661	1	1
9662	3M™ Surgical Clipper Charger for 9661	1	1
9667	3M™ Surgical Clipper Starter Kit 9667: 3M™ Surgical Clipper 9661 and 3M™ Surgical Clipper Charger for 9661 (clipper blade not included)	1	1
9690	3M™ Surgical Clipper Blade 9690 (for head and heavy body hair)	20	1

For more information about the 3M™ Surgical Clipper 9661 or other 3M Health Care products:

- Contact your local 3M sales representative
- Call the 3M Health Care Customer Helpline at 800-228-3957
- Visit [3M.com/infectionprevention](http://3M.com/infectionprevention)



\* Study EM-05-012664, Study EM-05-012713



## Medical and Surgical Requisites Pty Ltd

Call within Brisbane 07 3859 2900 33 Fulcrum Street Richlands Queensland 4077 [www.medsurg.com.au](http://www.medsurg.com.au)  
Phone 1300medsurg (1300 633 787) Email [medical@medsurg.com.au](mailto:medical@medsurg.com.au) Fax 07 3859 2995



# How to use

3M™ Surgical Clipper 9661

For use with 9660 blade

## Changing Blades



### Attach

Remove backing from the blade packaging, hold clipper body at a 45-degree angle and snap on single-use blade 9660.

NOTE: An audible snap indicates the blade is securely attached.




### Remove

To remove the blade, use your thumb to "pop off" the used blade into a sharps waste receptacle.

NOTE: Always use a new blade for each patient to avoid cross-contamination.


## Clipping

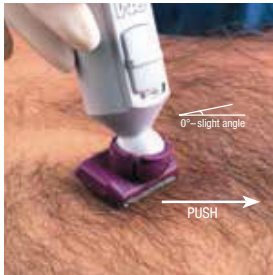

### Push

Position the clipper like a pencil with the clipper blade flat or at a slight angle on the skin surface. Gently push the clipper forward, against the direction of the patient's hair growth.

NOTE: Do not use excessive pressure to push the clipper down on the patient's skin.

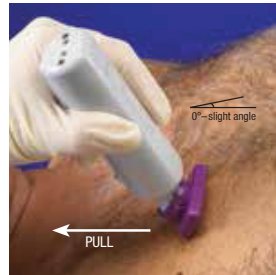

### Pull

Position the blade flat or at a slight angle on the skin surface and gently pull the clipper toward you.

NOTE: Do not rake the blade across the patient's skin.

## Cleaning



### Rinse

With the clipper uncorded and in the "off" position and the blade removed, rinse the handle under warm running water. Dry thoroughly before recharging or reusing.

NOTE: Do not submerge clipper body in any liquid solution



### Wipe

With the clipper in the "off" position and the blade removed, wipe the clipper handle with an alcohol or disinfectant wipe.

## Charging



### Charge

Before initial use, charge clipper for 18-24 hours. After each use, store the clipper handle in the "off" position in the plugged-in charge stand. Clipper provides approximately 100 minutes of cordless run time on a 12-14 hour recharge.

NOTE: There is no risk of overcharging, but after the initial use, charging for more than 14 hours will not result in longer run time.

Refer to package insert for warnings and more detailed instructions.

**3M**

**3M Health Care  
Infection Prevention Division**  
3M Center, Building 275-4E-01  
St. Paul, MN 55144-1000  
U.S.A.  
800-228-3957  
3M.com/infectionprevention

**Medical Division  
3M Canada**  
Post Office Box 5757  
London, Ontario N6A 4T1  
Canada  
800-563-2921

3M is a trademark of 3M.  
Please recycle. Printed in U.S.A.  
© 3M 2012. All rights reserved.  
70-2010-9135-5

## Medical and Surgical Requisites Pty Ltd

Call within Brisbane 07 3859 2900 33 Fulcrum Street Richlands Queensland 4077 www.medsurg.com.au  
Phone 1300medsurg (1300 633 787) Email medical@medsurg.com.au Fax 07 3859 2995



Established 1922

