



3M™ Avagard™ Antiseptic CHG Hand Rub Pump Reuse Guide

1

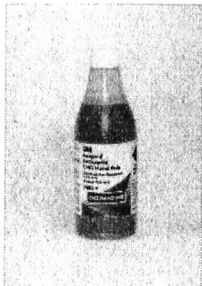

New bottle:
Place the new bottle on an even surface. Remove the transport cap from the new bottle.

2

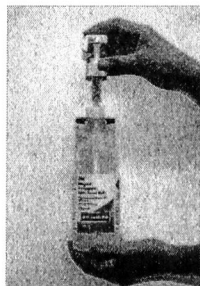

Empty bottle in bracket:
Unscrew the collar of the pump from the newly emptied Avagard Antiseptic CHG Hand Rub bottle.

Lift the pump directly up and out of the bottle, ensuring the stem does not touch the sides of the bottle or the wall bracket.

3



Continue holding the pump and ensure it does not come into contact with any surfaces.

Remove the bottle from bracket and dispose to appropriate waste stream.

4



Carefully place new bottle into the bracket.

5



Insert pump using a 'non-touch technique' into the new bottle, and screw the collar of the pump to tightly close.

Once the pump is inserted into the new bottle, cleaning of the pump head should be performed in accordance with hospital practices for cleaning medical equipment.

After inserting pump, wipe the head of the pump, and any other surfaces that may have been touched.

Use a damp, soft cloth and a hospital approved mild detergent, germicidal disposable wipes or disinfecting towelettes.

Allow to air dry.