

Recommendations on Personal Protective Apparel when Using Chemotherapy Drugs

CIRCUIT	GLOVES (pairs)	FILTER MASK	FACESHIELD OR GOGGLES	CAP	SHOE COVER	GOWN
Unpacking	2					•
Storage	1					•
Sterile preparation	2			•	•	•
Non-sterile preparation	1		*			•
Administration	1		*			•
Topical application	2		*			•
Waste management	1					•
Cleaning	1					•
Patient care	1		*			•
Spill, damaged or broken container	2	•	•		•	•
Cleanroom and antechamber cleaning	1			•	•	•
Exhaust cleaning	2	•	•	•	•	•
Handling other pieces of equipment, pharmacy, oncology and care unit/clinic	1					•
Handling bedding on floors/laundry	1					•

*If risk of splashing

GLOVE STYLES TESTED FOR USE WITH CHEMOTHERAPY DRUGS	PRODUCT CODE	CHEMOTHERAPY DRUGS TESTED																				
		BLEOMYCIN SULFATE	CARBOPLATIN	CARMUSTINE	CISPLATIN	CYCLOPHOSPHAMIDE	CYTARABINE	DACARBAZINE	DAUNORUBICIN	DOXORUBICIN HYDROCHLORIDE	ETOPOSIDE	5-FLUORO-URACILE	IDARUBICIN	IFOSPHAMIDE	MECHLOROETHAMINE	MELPHALAN	METHOTREXATE	MITOMYCINE	MITOXANTHRONE	PACLITAXEL	THIOTEPA	VINCRIStINE
Powder-Free Latex Examination Gloves																						
Micro-Touch® E.P.® Gloves	920155 / 920785	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•	•
Synthetic Powder-Free Examination Gloves																						
Micro-Touch® Affinity® - Neoprene	3770 / 3774		•	•	•					•	•	•				•		•		•		
Micro-Touch® Nitra-Touch® - Nitrile	6034010 / 6034014			▲	▲	▲				▲	▲	▲		▲						▲	▲	
Micro-Touch® Nitrile	4000 / 4008	•	•	•	•	•	•	▲	•	•	•	•	•	•	•	•	•	•	•	•	•	•
Micro-Touch® Nitrile E.P.®	6034050 / 6034054			•	•	•				•	•	•	•	•						•	•	•
Procedure Gloves																						
Micro-Touch® NitraTex® Sterile - Nitrile – EACH – PAIR	6034251 / 6034253 6034151 / 6034153			▲		▲				▲	▲	▲				▲				▲	▲	▲
Latex Powder-Free Surgical Gloves																						
No Powder®	860260 / 860785	•	•	•	•	•	•			•	•	•	•	•	•	•	•	•	•	•	•	•
Synthetic Powder-Free Surgical Gloves																						
Derma Prene® IsoTouch™ - Polyisoprene	20685255 / 20685290			▲		▲				▲	▲	▲				▲				▲	▲	▲
Derma Prene® Ultra - Neoprene	851155 / 851890	•	•	•	•	•	•			•	•	•	•	•	•	•	•	•	•	•	•	•

ASTM TESTING: F739: ● D6978: ▲

Tested for use with chemotherapy drugs

Ansell chemotherapy tested gloves have been tested either by ASTM F 739 or by ASTM D 6978. Specific chemical resistance data are available upon request by contacting our Customer Service Representatives. Gloves used for protection against chemotherapy drugs must be selected specifically for the type of chemicals used. Review material safety data sheets for the chemicals being used and determine the required level of protection.



Medical and Surgical Requisites Pty Ltd

Call within Brisbane 07 3859 2900 33 Fulcrum Street Richlands Queensland 4077 www.medsurg.com.au
 Phone 1300medsurg (1300 633 787) Email medical@medsurg.com.au Fax 07 3859 2995

