



Step One

Locate release button at the base of the unit



Step Two

Press button while gently pulling down front of unit, do not drop door



Step Three

Lift out empty 3M™ Avagard™ bottle with collar attached and dispose of thoughtfully



Step Four

Holding new bottle, place your thumb in indentation on cap and remove. Ensure the collar remains on around the tip of the bottle



Step Five

Place new product in dispenser



Step Six

Gently push bottle back in the dispenser until both bottle and dispenser 'clicks' into place

Lets keep a safe clean environment

Importance of maintaining your hand wash dispensers in good working order

Dispenser Maintenance Checklist:

- Outside surfaces and underneath each dispenser may be cleaned using, warm water with a mild disinfectant cleaner with a soft cloth (or as per Health Care facility Cleaning policy)
- Dry thoroughly after cleaning
- Deliver product onto wet hand - test hand hygiene dispenser is working well
- Replenish any empty dispenser with hand wash / hand care product - refer to refill Guide overleaf
- Make sure water/ product spills are mopped up on a regular basis - sinks / floors
- Report any breakages to your Supervisor
- Make sure you have clean hands when you finish



Medical and Surgical Requisites Pty Ltd

Call within Brisbane 07 3859 2900 33 Fulcrum Street Richlands Q 4077 PO Box 3965 Mt. Ommaney QLD 4074
Phone 1300medsurg (1300 633 787) Email medical@medsurg.com.au www.medsurg.com.au Fax 07 3859 2995

