


cT Intergrated E-Z Split Key Cup II Urine Test Drug Screen 6 Panel

Instructions for Use



1. Bring the pouch to room temperature before opening it. Remove the cup from the sealed pouch and use it as soon as possible.

2. Gently lift tab to open lid. Collect specimen in the cup. Cup should be filled to above minimum fill volume line. Secure cap by pressing down lid until it clicks.

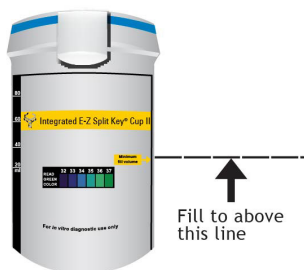

3. Check the temperature label up to four minutes after specimen collection. A green colour will appear to indicate the temperature of the urine specimen. The proper range for an unadulterated specimen is 32-38°C.

Temperature
Label

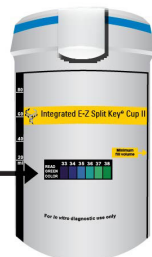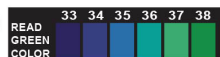


- Remove key from the lid. Place the cup on a flat surface. Push the key into the socket of the cup to begin the test. Start timer.




5. Check the cap for a tight seal. Date and initial the security seal label. Place it over the cap.

6. Remove the peel-off label covering the test results. Read the adulteration strip between three and five minutes.

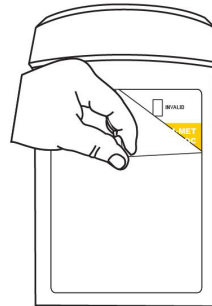


7. Compare the colours on the adulteration strips to the colour chart below. If the result indicates adulteration, do not interpret the drug test results. Either retest the urine or collect another specimen.


Adulteration Strip

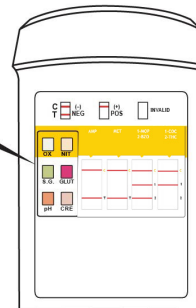

8. Read the drug strip results at five minutes.
The drug strip results remain stable for up to 60 minutes.

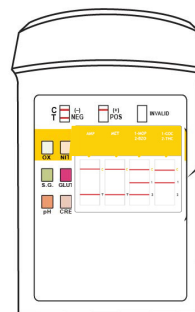

Drug Strip

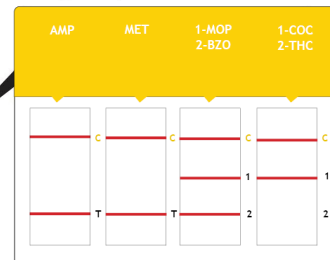

Adulteration Strip Colour Chart

OX PCC	Abnormal		Normal			Normal		Abnormal		NIT		
												
						20mg/dl	0mg/dl	50mg/dl	100mg/dl			
S.G.	Abnormal		Normal			Normal		Abnormal		GLUT		
												
	1.000	≥1.035	1.005	1.015	1.025	1.003						
pH	Abnormal		Normal			Normal		Abnormal		CRE		
												
	2	3	10	11	12	4	7	9	100mg/dl	20mg/dl	0mg/dl	10mg/dl