

OMRON

Forehead Thermometer MC-F300

Product Release Date:

November 2025

Product Release Sheet

Key Features



Distance Sensors

Distance sensors let you know that your thermometer is at the proper distance from you to support correct measurements.



Contactless measurement, eliminating the need for direct contact with your forehead.



Fast and accurate results in just one second, saving you both time and effort with every measurement.



With a simple point towards your forehead from up to 3 cm, the thermometer provides an accurate temperature reading instantly.



30-memory storage function, allowing you to conveniently store and recall previous temperature readings.



Ergonomic design, making it comfortable and easy to hold. The grip is designed to fit comfortably in your hand.



Flexibility to switch between Celsius (°C) and Fahrenheit (°F) temperature units.

Power Supply: DC 3.0 V [2 AAA alkaline batteries (included)]

2 year standard warranty plus additional 2 year warranty when registered online. (total 4 years)

Purchasing Information

- Size: 95mm (W) x 175mm (H) x 58mm (D)
- Weight: Approx. 230g
- Barcode: 4975479498343

Model: MC-F300

ARTG No. 302785



OMRON



Forehead Thermometer

MC-F300

Contactless
Fast Measurement
30-Memory Storage



Distance Sensor
for Accurate
Reading

For more information call us on 1800 807 464

Always read the label and follow the directions for use.

PTO for more info →


v10.25

JAD
J.A. DAVEY PTY LTD


Using the Thermometer

1 Gently pull off the sensor cover to remove it.

Measuring with the sensor cover on cannot measure accurately.

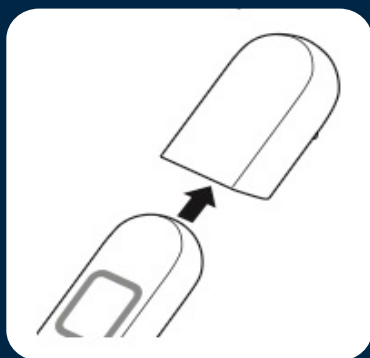

2 Press the [Power] button.

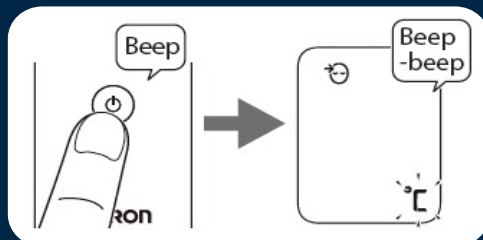

All indicators appear on the display for 3 seconds.

When "°C" (or "°F") flashes, and you hear two short beeps the device is ready for measurement.


Note:

If a reading is stored, the previous reading appears. Then, the device is ready for measurement.



3 Point the sensor toward the forehead.

When the sensor reaches the best distance for measuring (approx. 3 cm from the forehead), the short buzzer beeps twice and an OK symbol appears on the screen.

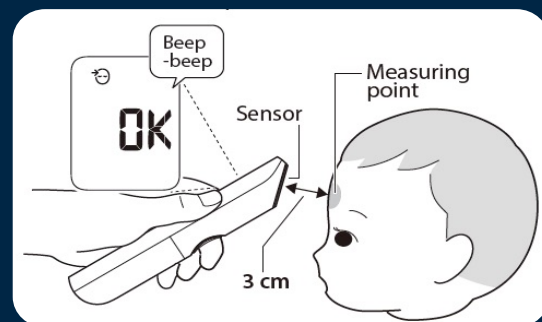


Note:

Always make sure the sensor is clean and undamaged and the forehead is clean with no sweat, cosmetics, scars, etc.

4 Press the [Measure] button.

The measurement is completed in 1 second with a long beep.

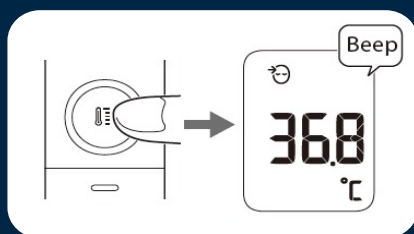

5 Check the measurement result.

After 1 second, "°C" (or "°F") flashes. Then, the device is ready for the next measurement.

To start a temperature measurement again immediately, repeat the steps from step 3.

6 Press the [Power] button.

The device turns off with a short beep.

The device automatically turns off if no button is pressed for 30 seconds.

Put on the sensor cover after measuring to protect the sensor.

Refer to 3.3 in product guide for Taking Surface Temperature (Surface Measurement Mode)

Intended Purpose

This device is a digital thermometer intended for use in measuring temperature from the forehead.