

PA5

Pediatric Audiometer



One hand operation



Interacoustics®

leading diagnostic solutions



Medical and Surgical Requisites Pty Ltd

Call within Brisbane 07 3859 2900
Phone 1300medsurg (1300 633 787)

33 Fulcrum Street Richlands Queensland 4077
Email medical@medsurg.com.au

www.medsurg.com.au
Fax 07 3859 2995



PA5

Pediatric Audiometer

One hand operation

The PA5 is a free field audiometer designed for trained healthcare professionals involved in testing young children. It features one hand operation, silent buttons and a variety of light and sound stimuli.

Conditioned Orientation Response

Typically, two PA5s are used for the COR test, one in each hand. This allows the tester to use either unit to evoke head turn and provide visual reinforcement. The single-hand controls on the PA5 give full control over frequency and intensity and make the test process quick, flexible and comfortable.

Extended battery life

The PA5 is powered by three standard AA batteries. It will turn off automatically after a period of inactivity and reactivate when a stimulation switch is touched.

Silent presentation buttons

The two stimuli, light and sound, are controlled silent buttons. Both stimuli can be presented simultaneously by letting one finger touch both buttons. This gives full freedom operating COR, VRA and APR tests.

Improved sensitivity

The PA5 includes white noise as well as warble and narrow band stimuli. White Noise has been shown to be especially useful for evoking responses in the early months.

Audiometric headphone

With children old enough to cooperate, it is possible to connect a single TDH39 headphone to the PA5 and obtain audiometric thresholds. With a headphone attached, the PA5 automatically adjusts to the correct calibration for the TDH39.

Additional features

- Warble Tone
- Narrow Band Noise
- Flashing Light Stimulation
- TDH39 Audiometric Headphone (Optional)
- Robust, Lightweight
- Standard AA Batteries
- Automatic Battery Switch
- Automatic Battery Indication

Tests

- *Conditioned Orientation Response (COR)*
- *Visual Reinforcement Audiometry (VRA)*
- *Aurolpalpebral Reflex (APR)*



Established 1922

Medical and Surgical Requisites Pty Ltd

Call within Brisbane 07 3859 2900 33 Fulcrum Street Richlands Queensland 4077 www.medsurg.com.au
Phone 1300medsurg (1300 633 787) Email medical@medsurg.com.au Fax 07 3859 2995

