

Recommended Safe Handling Procedures

- Always ensure the lid is closed and tight before moving
- Always carry the container by the handle provided - do not hold the container body
- Always inspect container for signs of penetration before and after handling or transporting
- Do not subject the container to unnecessary shocks or vibration
- If the container is dropped inspect it carefully for damage then dispose of it properly

Pen Needle Removing Feature - Instructions for Use



1. Insert pen device

Hold pen vertically and push needle into the remover until it engages.



2. Remove needle

Whilst the needle is held securely, unscrew the pen in an anti-clockwise direction. The needle will disconnect and drop into the container.



A pen needle-removing feature is built-in to the safety flange for easy, one-handed removal of needles from an insulin pen device

TIP: Never overfill containers past the maximum fill line.

Single Use Sharps Containers

Sharps Containers are for discarding all contaminated sharps waste including used hypodermic needles, syringes, scalpels, lancets and devices used in medical, surgical and home healthcare environments.

- Designed with safety in mind
- Manufactured from high quality, puncture resistant material which can be incinerated

**When safety matters,
think TERUMO**