



Haemolance Plus® Single Use Safety Lancet

1. Prepare the patient and gain consent.
If possible wash the digit, especially if physically dirty, this also helps to warm the digit.
2. To remove the protective cap, simply twist the cap and pull straight out.
3. Place the lancet between the first and second fingers while resting the thumb on the button. Place the lancet gently against the side of the finger instead of the tip to minimise patient discomfort. Press the button.
4. Gently apply intermittent pressure near the puncture site to obtain the required blood volume.
5. Safely dispose of the used lancet into a sharps container.



When safety matters,
think TERUMO



Established 1922

Medical and Surgical Requisites Pty Ltd

Call within Brisbane 07 3859 2900 33 Fulcrum Street Richlands Q 4077 PO Box 3965 Mt. Ommaney QLD 4074
Phone 1300medsurg (1300 633 787) Email medical@medsurg.com.au www.medsurg.com.au Fax 07 3859 2995

