

Surgidry Dispenser Loading Instructions

It has been brought to our attention, that our Surgidry Interleaved Towels may be getting caught in the dispenser.

The manufacturer has put the label sleeve on the towel upside down and made the towel wider.

To correct this (until our next batch arrives), we recommend loading the paper upside down which will load the paper from the back to help run smoothly.

Please see image below.

We apologise for any inconvenience.



Correct way to load towels in the interim.



Medical and Surgical Requisites Pty Ltd

Call within Brisbane 07 3859 2900 33 Fulcrum Street Richlands Queensland 4077 www.medsurg.com.au
Phone 1300medsurg (1300 633 787) Email medical@medsurg.com.au Fax 07 3859 2995

